


# Precise Quick Start Guide

## Parts of the Hearing Aid

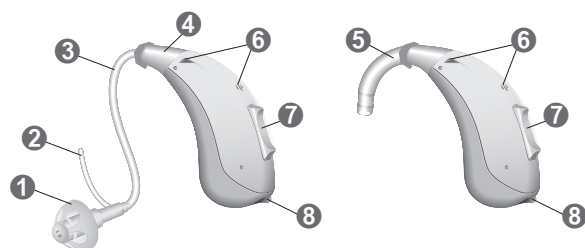

- |                             |                                       |
|-----------------------------|---------------------------------------|
| 1 Ear piece (Dome)          | 5 Earhook                             |
| 2 Retention tail (optional) | 6 Microphone openings                 |
| 3 Slim Tube                 | 7 Rocker switch (control)             |
| 4 Adapter for ThinTube      | 8 Battery compartment (on/off switch) |

## Turning the hearing aid ON/OFF

To turn the hearing aid on, close the battery compartment.


Open the battery compartment to turn off the hearing aid.



## Rocker Switch

Short press: Volume Louder  
Long Press >3 seconds: Program Up

Short press: Volume Down  
Long Press >3 seconds: Program Down

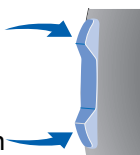



## Low Battery Indicator

When the battery is low the sound will become weaker or you will hear a descending scale every few minutes until the battery dies.

## Insertion/Removal of Battery


Remove the protective foil from the new battery. Wait for two minutes before inserting to activate the battery.

Open the battery compartment fully using your finger. Pull out dead battery. Insert the new battery with the "+" symbol facing up. Carefully close the battery door.

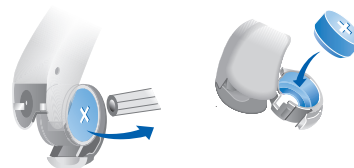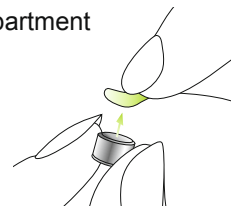

## Identifying the hearing aid for the right and left ear

Side indicator:  
Right = Red  
Left = Blue

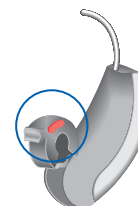

## Hearing Programs

### Program 1: Universal

The best choice for most listening situations. It is always active when you first switch on your hearing instrument.

### Program 2: Noisy Environment

For crowded places with lots of background noise.

### Program 3: Music



For listening to music.

### Program 4: Phone Acoustically


For phone calls with standard phones.

## myControl app

Did you know you can control your hearing aids with the myControl app?  
First download the app, which is available for Android and iOS.

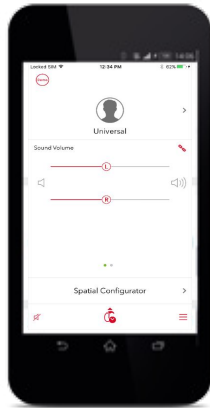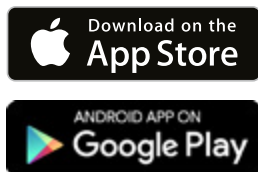

Turn your smartphone into a discreet remote control for your hearing aids. The myControl App™ lets you enjoy a high level of handling comfort right at your fingertips.

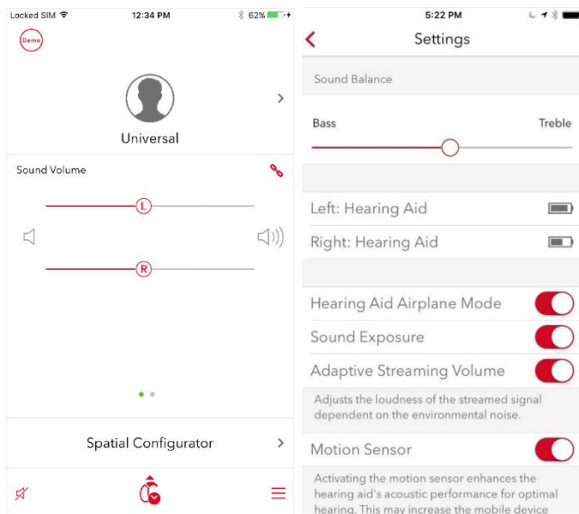

**Program and volume changes**  
Adapt to changing listening situations by making easy and discreet adjustments.

**Bass and treble**  
Use SoundBalance™ to effortlessly change the tone of your environment according to your listening preference.

**Remote Microphone**  
This feature turns the iOS mobile device into a remote microphone

**The myControl app can to so much more!** Consult the Precise User Guide or see our website for more information:  
[www.PreciseHearingCare.com](http://www.PreciseHearingCare.com)

## App download.

### App-solutely easy to get!

Easy to use and easy to get: Simply download the myControl App to your Android or iOS smartphone on the Google Play Store or Apple App Store. The best part is the download is free!

#### Step 1



Make sure you have a Google Play Store account for your Android phone or an Apple App Store account for your iPhone.

#### Step 2



Log into your Google Play Store or Apple App Store account with your Android or iOS smartphone. Find the myControl App.

#### Step 3



Follow the instructions to install the app on your smartphone.

#### Step 4



Discover how easy it is to control your hearing instruments by simply trying your new app.

